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**DIRECTIONS FOR USE**

When you first get your lenses, be sure that you are able to put the lenses on and remove them (or have someone else available who can remove the lenses for you) before leaving your Eye Care Professional's office.

DO NOT use if the sterile blister package is opened or damaged or after the expiry date.

**Step 1: Getting Started**

It is essential that you learn and use good hygiene in the care and handling of your new lenses. Cleanliness is the first and most important aspect of proper contact lens care. In particular, your hands should be clean, dry and free of any soaps, lotions, or creams before you handle your lenses.

Before you start:

- Always wash your hands thoroughly with warm water, a mild soap, rinse carefully and dry with a clean lint-free towel before touching your lenses to reduce the chance of getting an infection.
- You should avoid the use of any soaps containing cold cream, lotion, or cosmetics before handling your lenses. These substances may come into contact with the lenses and interfere with successful wearing. It is best to put on your lenses before putting on makeup.

**Step 2: Opening the Packaging**

Always confirm the lens parameters (e.g. diameter (DIA), base curve (BC), lens power (D), etc.) printed on the multi-pack and on the individual lens package match your prescription. DO NOT use if there is a mismatch.

**Multi-pack**

Each multi-pack contains individually packaged lenses. Each lens comes in its own foil-sealed plastic package designed specifically to keep it sterile while sealed.

**Lens Package**

To open an individual lens package, follow these simple steps:

1. Shake the lens package and check to see that the lens is floating in the solution.

2. Carefully peel back the foil closure to reveal the lens.

3. Place a finger on the lens and slide the lens up the side of the bowl of the lens package until it is free of the container. Occasionally, a lens may stick to the inside surface of the foil when opened, or to the plastic package itself. This will not affect the sterility of the lens. It is still perfectly safe to use. Carefully remove and inspect the lens following the handling instructions.

**Lens Handling Tips**

- Handle your lenses with your fingertips, and be careful to avoid contact with fingernails. It is helpful to keep your fingernails short and smooth.
- After you have removed the lens from the packaging, examine it to be sure that it is a single, moist, clean lens that is free of any nicks or tears. If the lens appears damaged, DO NOT use it.

**Step 3: Placing the Lens on the Eye**

Once you have opened the lens package and examined the lens, follow these steps to apply the lens into your eye:

1. BE SURE THE LENS IS NOT INSIDE-OUT by following one of the following procedures:

- Place the lens on the tip of your index finger and check its profile. The lens should assume a natural, curved, bowl-like shape. If the lens edges tend to point outward, the lens is inside out.
- Gently squeeze the lens between the thumb and forefinger. The edges should turn inward. If the lens is inside out, the edges will turn slightly outward.
- Place the lens on the tip of your index finger and, looking up at the lens, locate the numbers 1-2-3. 1-2-3 indicates correct orientation, while a reverse of 1-2-3 indicates the lens is inside out. If the lens is inside out (reverse 1-2-3), invert the lens and locate the numbers again to confirm correct lens orientation. Note that the 1-2-3 marking is not present on all ACUVUE® lenses.

2. With the lens on your index finger, use your other hand to hold your upper eyelid so you won't blink.

3. Pull down your lower eyelid with the other fingers of your "applying" hand.

4. Look up at the ceiling and gently place the lens on the white of the lower part of your eye.

5. Slowly release your eyelid and close your eye for a moment.

6. Blink several times to centre the lens.

7. Use the same technique when applying the lens for your other eye.

There are other methods of lens placement. If the above method is difficult for you, ask your Eye Care Professional for an alternate method.

**Step 4: Centring the Lens**

A lens, which is on the cornea (clear dome at the front of the eye), will very rarely move onto the white part of the eye during wear. This, however, can occur if application and removal procedures are not performed properly. To centre a lens, follow either of these procedures:

- Close your eyelids and gently massage the lens into place through the closed lids.

OR

- Gently move the off-centred lens onto the cornea (centre of your eye) while the eye is opened using a clean finger pressure on the edge of the upper lid or lower lid.

**Cosmetic Lenses and Visual Symptoms**

Cosmetically tinted contact lenses may let less light through than non-cosmetic lenses. Therefore, you may experience some visual symptoms while wearing them (i.e. seeing the lens pattern in your peripheral vision).

**Hazardous Conditions**

- If you use aerosol (spray) products, such as hair spray, while wearing lenses, keep your eyes closed until the spray has settled.
- Avoid all harmful or irritating vapours and fumes while wearing lenses.
- Never rinse your lenses in water from the tap. Tap water contains many impurities that can contaminate or damage your lenses and may lead to eye infection or injury.

**Water Activity**

- Do not expose your contact lenses to water while you are wearing them.

**Lubricating/Rewetting Solutions**

- Your Eye Care Professional may recommend a lubricating/rewetting solution for your use. These solutions can be used to wet (lubricate) your lenses while you are wearing them.
- Do not use saliva or anything other than the recommended solutions for lubricating or rewetting your lenses. Do not put lenses in your mouth.

**Sharing Lenses**

- Never allow anyone else to wear your lenses. Sharing lenses greatly increases the chance of eye infections.

**Adhering to the Prescribed Wearing & Replacement Schedules**

- Never wear your lenses beyond the amount of time recommended by your Eye Care Professional.

**Step 1: Getting Started**

It is essential that you learn and use good hygiene in the care and handling of your new lenses. Cleanliness is the first and most important aspect of proper contact lens care. In particular, your hands should be clean, dry and free of any soaps, lotions, or creams before you handle your lenses.

Before you start:

- Always wash your hands thoroughly with warm water, a mild soap,

rinse carefully and dry with a clean lint-free towel before touching your lenses to reduce the chance of getting an infection.

- You should avoid the use of any soaps containing cold cream, lotion,

or cosmetics before handling your lenses. These substances may come into contact with the lenses and interfere with successful wearing. It is best to put on your lenses before putting on makeup.

It is best to put on your lenses before putting on makeup.

It is best to put on your lenses before putting on makeup.