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or parameters without consulting your ECP. Ask your ECP about the follow-up visit schedule.

• **ALWAYS CONTACT ECP BEFORE USING ANY MEDICINE IN YOUR EYES.**

• **NEVER use tweezers or other tools to remove your lenses from the lens container.**

• **Smoking and/or taking some medicines (like contraceptives or other medicines) may increase eye problems or eye discomfort.**

• **AVOID all harmful or irritating vapors/fumes. Keep your eyes closed during usage of aerosol products.**

ADVERSE REACTIONS: Burning, stinging, itchy, and/or dry eyes - Reduced lens comfort or feeling of something in your eye - Swelling or inflammation in or around the eyes - Eye redness - Eyelid problems - Watery eyes and/or unusual eye secretions - Poor or blurred vision - Rainbows or halos around objects - Sensitivity to light - temporary impairment due to peripheral infiltrates, peripheral corneal ulcers, and corneal erosion. If you notice any problems, you should IMMEDIATELY REMOVE YOUR CL and contact your ECP.

DIRECTIONS FOR USE: Not following directions for use may result in severe eye problems.

• **ALWAYS WASH YOUR HANDS BEFORE TOUCHING YOUR CL as CL are very sensitive to contamination.**

• **NEVER PUT YOUR LENSES IN YOUR MOUTH OR PUT SALIVA ON YOUR LENSES.**

• **NEVER ALLOW ANYONE ELSE TO WEAR YOUR LENSES.**

• **Follow your ECP instructions regarding wear and removal of your CL.**

Lens Package: Carefully peel back the foil closure to reveal the lens.

Lens Insertion: Ensure the lens is not inside out. Place on your index finger to verify the shape. The lens should assume a natural, curved, bowl-like shape (Fig.1). With the lens on your index finger, use your other hand to hold your upper eye lid so you will not blink. Place the lens onto the eye. (Fig 2 & 3)

Lens Removal: Look up, slide the lens to the lower part of the eye using the forefinger. Gently pinch the lens between the thumb and forefinger to remove the lens. (Fig.4)

EMERGENCIES: If chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into your eyes: FLUSH EYES IMMEDIATELY WITH TAP WATER AND IMMEDIATELY CONTACT YOUR ECP OR VISIT A HOSPITAL EMERGENCY ROOM RIGHT AWAY.

REPORTING OF ADVERSE REACTIONS (Side Effects): Any incident experienced whilst wearing ACUVUE® Brand CL should be reported to the manufacturer and/or its authorized representative and/or to your national authority.

The following symbols may appear on the labels or packaging:

Symbol	Definition
	Date of Manufacture
	Use-by Date (expiration date)
	Do Not Use if Package is Damaged
	Do Not Re-Use
	Consult Instructions for Use or Consult Electronic Instructions for use. www.e-ifu.com
	Caution
	CE-mark and Identification Number of Notified Body
DIA	Diameter
BC	Base Curve
D	Dioptrre (lens power)
CYL	Cylinder Power
AXIS	Axis
MAX ADD	Highest Near Addition That Can Be Corrected
LOW / L	"Low" near ADD
MID / M	"Medium" near ADD
HGH / H	"High" near ADD

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